



Endwarts Freeze

Uses

Uses the cryotherapy method to remov warts and verrucas from hands and feet. Suitable for adults and children over 4 years. One treatment is usually sufficient to remove the wart or verruca.

Dosage

Adults and children over 4 years: turn the white part one full turn in the direction of the arrow until the star is back in the shaded zone. Push down the white part for exactly 2 seconds. Place upright on a table with the transparent part down. Hold the device with a steady grip and with the other hand, push the white part downward for exactly 2 seconds. Remove the cap. If there is no ice vapour, repeat step 2. Immediately press the tip firmly against the wart for 15 seconds on hands, fingers and arms and for 40 seconds on feet.

Format and Ingredients

Freezing agent containing:
N20 gas,

Category:
Footcare, Skin Care and Antiseptics

Manufacturer:
VIATRIS UK HEALTHCARE LTD

Pack size: 7.5g

RRP: £19.99

Legal Status: MD