



Scholl Freeze Verruca & Wart Remover

Uses

For the removal of verrucae and warts.

Dosage

Adults and children over 4 years: following the instructions provided, attach an applicator bud to the white plastic cap. Keeping the can upright press the cap down firmly for five seconds and then remove cap and applicator bud. Wait for 15 seconds, holding the bud pointing downwards, and then apply to the centre of the verruca or wart for a minimum of 30 seconds but no longer than 40 seconds. Do not dab. Discard the applicator after a single use. Each can comes with 12 disposable applicators. Under 4 years: not recommended.

Format and Ingredients

Aerosol spray

80ml

99

us: MD