



Bazuka Advance Treatment Pen

Uses

For the treatment of stubborn warts and verrucas.

Dosage

Adults and children over 8 years: Use treatment every other day (on days 1, 3 and 5) followed by 2 days' rest. Can be used for a maximum of 6 weeks. Soak the affected area in lukewarm water for at least 5 minutes to soften the skin, prior to treatment. Gently rub any hard skin off the surface of the wart or verruca, taking care not to break the skin or file the wart or verruca itself. Remove the cap of the pen by pressing down and turning counter clockwise. Gently stroke the wart with the tip of the treatment pen for the appropriate contact time (see instruction leaflet for details). On application, the treatment may not be visible as it is immediately absorbed into the wart or verruca. Treat the whole wart or verruca but take care not to apply product onto healthy skin. Recap the pen firmly and wash hands after use.

Format and Ingredients

Pen containing:
Formic Acid 85%,

Category:
Footcare

Manufacturer:
DIOMED DEVELOPMENTS LTD

Pack size: 3ml

RRP: £15.00

Legal Status: MD